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Commander l'écouvillon

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<https://shop.embarkvet.com/products/embark-for-breeders-dog-dna-test-kit>

CYBER MONDAY SALE GOING ON NOW

Embark for Breeders Dog DNA Test Kit

~~\$149~~ **\$129 Sale**

Or 3 interest-free payments of \$43 with [affirm](#). [Learn More](#)

Quantity: **Add to cart**

Cyber Monday Sale:

- 1 Kit: \$129 per kit (save \$20)
- 2-19 Kits: \$119 per kit (save \$30)
- 20+ Kits: \$115 per kit (save \$34)

Find out why thousands of breeders have trusted Embark to help manage their breeding program, from improving genetic health and diversity to screening for disease mutations, understanding coat traits, and more.

An Embark for Breeders dog DNA test kit provides breed-relevant disease screening for your purebred dogs, insights on their health, traits (including coat color, body size and more), Genetic inbreeding coefficient and results for nearly all genetic mutation tests recommended by national breed clubs are included in every kit. Our test results are accepted by the Orthopedic Foundation for Animals (OFA) for all conditions where OFA has an established DNA registry.

Product ships free in the continental U.S. Most packages arrive within 3-5 business days. Expedited shipping options available. Return shipping of swabs included.

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Do More

Get your whole pack involved or give someone the gift of discovery with Embark.

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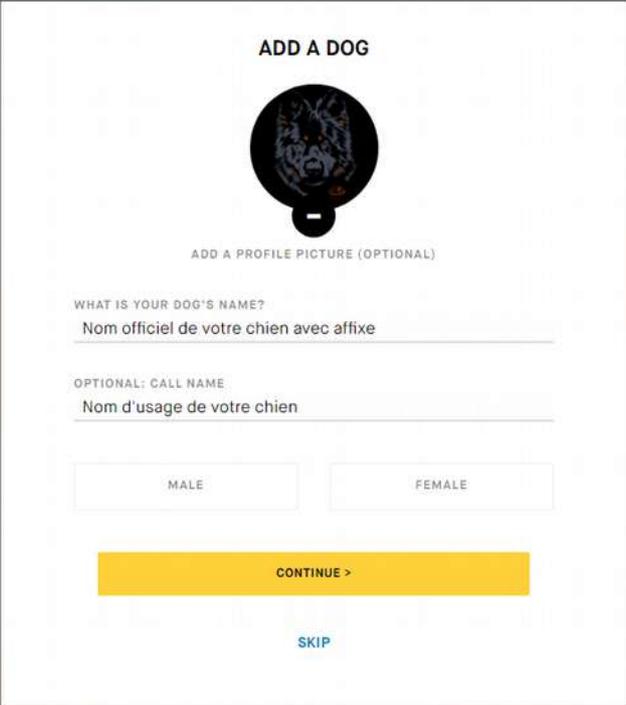
DATE OF BIRTH, KNOWN OR ESTIMATED (OPTIONAL)

Select: Day Month Year

If you don't know the exact birthdate, you can just enter the year, or the nearest year and month.

LOCATION (OPTIONAL)

This information will help us to highlight other Embark dogs that may be of interest to you.

Where does this dog currently live? 

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This location is a: Home Breeder Shelter Other

Please choose Home, Breeder, Shelter or Other

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SEX

MALE FEMALE

IS SIRIÛS ALREADY NEUTERED OR EXPECTED TO BE NEUTERED?

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BREED

German Shepherd Dog

Don't see the right breed here? Add it to your [Kennel Profile](#).

MAIN REGISTRATION ORGANIZATION (OPTIONAL)

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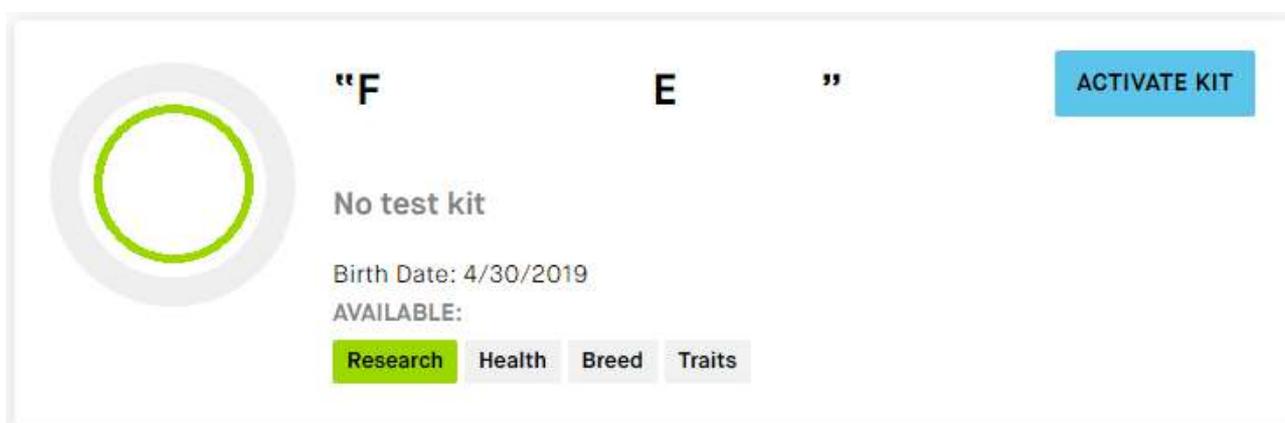
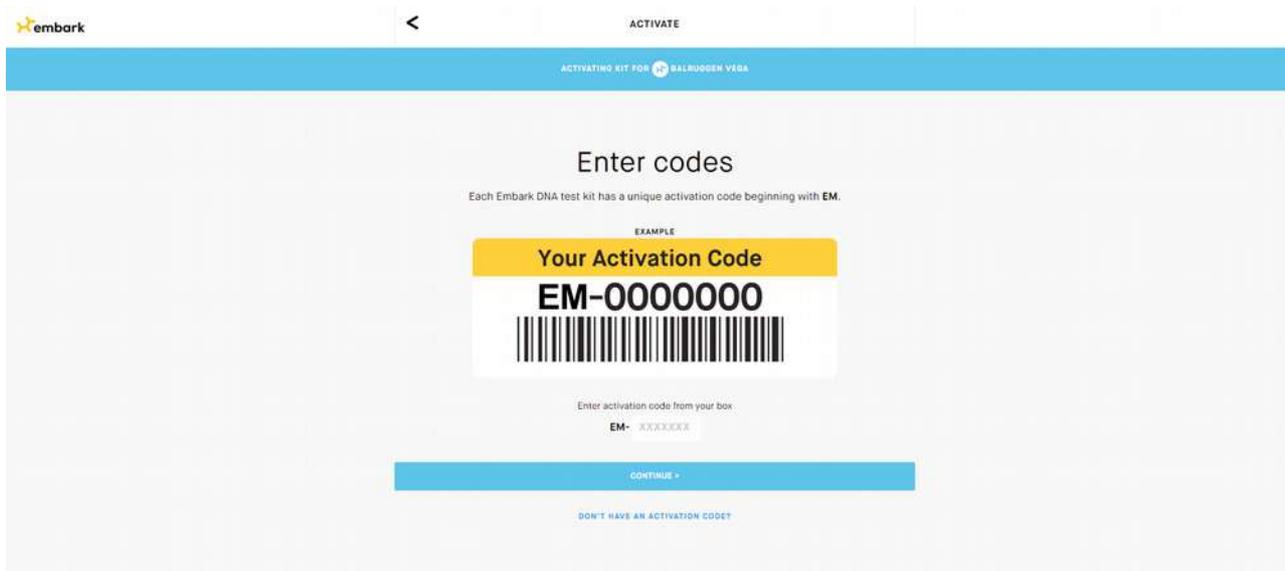
SWAB TUBE NUMBER

S1020061506564

I want this dog's data to contribute to medical and scientific research

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ACTIVATING KIT FOR BALRUGGER VEGA

Text when ready?

When the results are ready we will notify you by email at lestloupdargoat@gmail.com

RECOMMENDED: You can also provide a mobile phone number to receive a text message when the results are ready:

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OFA SUBMISSION REPORT

Send to the OFA >

You will first see an online preview; then click 'Download / Print' in the top-right to download a PDF file that you can either print out and mail to the OFA, or submit electronically via email to the OFA.

Important: Your submission will need to include payment to the OFA, and a completed [OFA DNA application form](#). At the top of the application, please be sure to write which tests you want to register with the OFA.

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GENETIC STATS

Predicted adult weight:

Based on the date of birth you provided

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